< relaxmyway

NECK MASSAGER II

< relaxmyway

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< relaxmyway

Since being established in 2000, the name CASADA stands for high quality products on the health-spa and fitness market. Today, CASADA-equipment is available in 37 countries world-wide.

Constant further development, an unmistakable design and a focused orientation towards a better quality of life are what characterize CASADA-products, as much as their unbeatable and reliable quality.

Congratulations!

With the purchase of this massage device you have shown awareness of your health.

In order for you to enjoy the advantages of your massage device in the long term, we ask you to read and adhere to the instruction manual carefully.

We wish you a lot of fun with your personal Neck Massager II.

Table of contents

10	Safety instructions
10	Inside the package
10	Start up
11	Technical data
11	Specials/Equipment
12	Usage
12	Control panel
13	Troubleshooting
13	Storage/Care
13	Conformity declaration
13	Warranty conditions

Safety instructions

Please carefully read the following instructions before using this massage device to ensure its proper function and optimal effect. Please retain these instructions for use!

- This massage device complies with the recognised principles of technology and current safety regulations.
- This equipment is maintenance free. Any necessary repairs must be performed by an authorised professional. Improper use and unauthorised repairs are prohibited for safety reasons and will void the warranty.
- Never touch the power plug with wet hands.
- Please avoid the equipment coming into contact with water, high temperatures and direct sunlight.
- Be sure to unplug the equipment from the power point during extended periods of non-use to eliminate the risk of a short circuit.
- Do not use damaged plugs, cords or loose sockets. Damaged plug or cords must be replaced by the manufacturer, service representative or qualified personnel. In the event of malfunction unplug the equipment immediately. We assume no liability for damages resulting from wrongful or improper use.
- In order to prevent excessive stress on muscles and nerves, the recommended massage time of 30 minutes should not be exceeded.
- Never use pointy or sharp objects.
- Any massage – also massage by hand – has to be avoided during pregnancy and/or in case of one or more complaints in the area of the massage: recent injuries, thrombotic illnesses, inflammations and swelling of any kind as well as cancer. Consultation of your physician is recommended prior to any treatment of afflictions and ailments.
- Do not use the device while driving.
- If you depend on electrical aids such as a pace maker, please consult your physician prior to usage in any case.

Inside the package

- Neck Massager II
- Adapter
- 12V Car adapter

Start up

Please connect the adapter with the device and subsequently plug the power plug into the power socket. Now turn on the device by the control panel.

With the car adapter the Neck Massager II can also be used when travelling (not to be used when driving!). Simply plug the car adapter in the cigarette lighter.

Technical data

Product measurements:	41 x 52 x 15,5 cm
Weight:	1,5 kg
Power source:	input: AC 100-240V ~ 50/60Hz output: DC 12V, 2A
Rated power:	24 Watt
Automatic runtime:	15 minutes
Certificates:	   

Specials / Equipment

KNEADING

Skin and muscles are either clasped and kneaded between the thumb and the index finger or with both hands. This massage technique is used in particular to ease tension.

HEATING

A deep penetrating and heat-radiating effect in which one can perfectly relax. This function is particularly beneficial for muscle tension due to the deep reaching effect.

REVERSE

The reverse function changes the direction of the massage heads.

AUTOMATIC

Specifically developed automatic programmes for the whole body, which offer a big range of different massage techniques. Additionally you can also turn on the heating function, if you'd like.

CAR ADAPTER

The handy car adapter makes your massage device portable.

For your own safety we equip the machine with a protection against overheating. Under high load and pressure, heat is generated in the motors. Before this can be a source of danger for you the machine will switch off automatically after 15 min. After a waiting time between 30 and 40 minutes you can turn it on again.

Usage

Place the Neck Massager II in your neck and let the loops fall forward on your chest. Use your hands to grasp the loops on the right and to the left and locate the massage device in a comfortable way (Illustration 1).

Now your personal Neck Massager II is ready for use. Turn the power on [⏻] the 15-minute automatic programme starts. The massage heads changes direction by 1 min. The heating function is switched on from the start, but can be switched off at any time [HEAT]. If you prefer the other massage direction you can do it by using the arrow keys. In this case the automatic programme will be interrupted and the device switches to manual mode.

Alternatively, the device can also be used to massage the thigh (Illustration 2).

Illustration 1

Illustration 2

Control panel

- [1] Power on/off
- [2] **ARROW KEYS**
Pressing one of the arrow keys a 15-minute massage starts. To change the massage head direction, press the other arrow button.
- [3] **HEAT** Switch on/off the heating function
- [4] **AUTO** Turn on/off the 15 minute automatic programme

Troubleshooting

Error	Adjustment
The massage device cannot operate even though it has been connected to the power supply.	<ul style="list-style-type: none">• Check the connection of the adapter.• The device could be overheated after prolonged use. It will automatically cool down to resume its normal condition. Please turn off the power and allow the device to cool off for 30 to 40 minutes.

Storage / Care

Regular care will extend the life of the device whilst also ensuring safe operation at all times.

1. Recommended storage: dry, free of dust.
2. Keep liquids and open fire away from the device.
3. Clean the equipment using a dry, soft cloth.
4. Always unplug the device when not in use.

Conformity declaration

Corresponds to the European guidelines EMC 2014/30/EU, LVD 2014/35/EU.

Warranty conditions

The dealer grants the statutory warranty period on the product described overleaf. The warranty applies from the date of purchase. The date of purchase is demonstrated by the sales receipt.

The dealer is to repair or replace products registered within Germany which are found to be defective free of charge. Parts subject to wear and tear such as cloth covers are not covered. To assert your claim, simply return the defective product with the sales receipt before the end of the warranty period.

The warranty claim does not apply if it is established that a defect has occurred e.g. through external influence or as a result of repair or alteration not carried out by the manufacturer or an authorised dealer.

The warranty granted by the seller is limited to repair or replacement of the product. Under this warranty, the manufacturer or seller has no further liability and is not responsible for damage caused by disregard of the operating instructions and/or improper use of the product.

If inspection of the product by the seller reveals that the warranty claim relates to a defect not covered by the warranty or that the warranty period has expired, the cost of the inspection and repair are to be borne by the customer.

Casada International GmbH
Obermeiers Feld 3
33104 Paderborn, Germany
info@casada.com | www.casada.com

Copyright Images and texts are subject to the copyright of Casada International GmbH and may not be used for other purposes without express confirmation.
Copyright ©2017 Casada International GmbH. All rights reserved.

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Copyright ©2017 CASADA International GmbH.
Copyright © 2017 CASADA International GmbH.
Copyright ©2017 Casada International GmbH. Wszystkie prawa zastrzeżone.