pekatherm
warm softness

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IMPORTANT INFORMATION! PLEASE, READ CAREFULLY AND KEEP FOR FUTURE REFERENCE!

THESE INSTRUCTIONS PROVIDE INFORMATION AND ADVICE ON INSTALLATION, USE AND MAINTENANCE OF THE PRODUCT. READ THEM CAREFULLY BEFORE USING THE PRODUCT TO ENSURE SAFE AND PROPER USE. NON OBSERVANCE OF THESE INSTRUCTIONS CAN CAUSE PERSONAL INJURIES, AS WELL AS PRODUCT DAMAGE OR DESTRUCTION. KEEP THESE INSTRUCTIONS AND PASS THEM ON WITH THE PRODUCT IF SOLD OR LENT TO OTHERS.

PRODUCT PARTS

(see pic. 1, 2, 3)

- a. Flexible heating part (pad or underblanket)
- b. Control unit
- c. Power cord
- d. Cloth cover (only included with heating pads)
- e. User manual
- f. Detachable control unit (according to model)
- g. Temperature level LED indicators
- h. Temperature selection button

BEFORE USE

1. After removing the packaging, inspect the product carefully to make sure it has no signs of damage caused during transport. In case of any doubt, do not use the product and take it to your dealer or to an authorized service centre.
2. Before plugging the product in make sure your mains' specifications correspond to those of the product stated on the label sewn into the heating part or, in the case of PVC pads, printed directly on the surface.
3. If your product has a detachable control unit, only use the control unit provided with the product, the reference of which is also stated on the label sewn into the product or printed on its surface. Do not attempt using control units from other appliances.
4. This product must not be used with sick or disabled persons, with persons insensitive to heat or persons with restricted physical, sensory or mental capabilities. It also must not be used by children or by people lacking the required experience and knowledge of product operation unless they are properly instructed on how to operate the product safely, or unless the control unit is switched on and set by a responsible person. In all circumstances, while using the product, children should always be supervised by an adult to prevent them from playing with it.
5. Do not use the product under the effects of alcohol, sleeping pills or sedatives.
6. Do not use the product on animals.

7. Do not apply the product to any areas of the body where the skin presents scars, wounds, inflammation or any other alteration due to illness. Consult your physician.
8. This product is designed for home use only. It is not intended for professional applications or medical use in hospitals.
9. The heating pad must only be used for warming the human body. The heating underblanket must only be used for warming beds. Make sure your underblanket is totally unfolded and covered with a bed sheet. Any other use of the product will be considered hazardous. The manufacturer will decline all responsibility for any damage due to improper or incorrect use.
10. Pacemaker users should consult their physician before using the product.
11. Do not use the product folded or creased. Be careful with your underblanket when used on adjustable beds or twin mattresses. Do not store the product creased.
12. Do not stick pins, needles or any other sharp objects into the product.
13. Do not place heavy objects on the product. When your underblanket is switched on, avoid putting heavy objects on the bed.
14. Never bring the product into contact with other sources of heat, such as other heating pads, underblankets, hot water bottles, etc.
15. Do not cover the control unit while operating the product; do not place it on the heating element or under it while the unit is in operation.
16. Excessively long use of the heating pad running at a high temperature may produce skin burns. Avoid falling asleep with the heating pad in operation.
17. Do not leave the product unattended while in operation.
18. In any case avoid wetting the control unit.
19. Do not touch the control unit with wet hands while the product is plugged in.
20. Do not use the product in excessively humid environments or in environments where the risk of splashing liquids on it is high (e.g. bathrooms, showers, swimming pools). In the presence of bottles or glasses with beverages, take care not to splash them on the control unit.
21. Do not expose the product to environmental agents (rain, sun, etc.).
22. Unplug the product from the mains when not in use.
23. Before cleaning the product make sure it is unplugged from the mains.
24. Cleaning instructions are found in a specific section of this manual. To protect the heating pad from dust or damage the use of the cotton cover is recommended.
25. Protect the power cord from scraping against sharp edges and from coming into contact with very hot objects that could damage it.
26. Do not pull on the power cord or use the heating part itself to pull and unplug the product from the mains.
27. Do not hold the product by the power cord to move or carry it.
28. Inspect the product frequently for signs of wear or damage. If these signs appear, do not use the product and take it to an authorized service centre.
29. This product has no parts intended to be repaired or replaced by the user. If any part of the product fails, or the product does not perform properly, or there is any other kind of anomaly, do not attempt to repair it and instead take the product to an authorized service centre.
30. The appliance must be returned to the manufacturer or his agent if the fuse of control unit has ruptured.
31. For any further information, please consult our customer service.

INSTRUCTIONS

HEATING PADS

“U” line pads: Rectangular heating pads, to be applied to any part of the body

“S” line pads: Specially shaped pads for targeted treatment, to be applied to specific body parts

MODEL	MEASURES	FINISH	DETACHABLE	APPLICATION	POWER
U20T	40x30cm	Cloth	√	Universal	110W
U20P	40x30cm	PVC	-	Universal	110W
U30T	47x39cm	Cloth	√	Universal	120W
U30P	47x39cm	PVC	-	Universal	120W
S20	45x25cm	PVC	-	Neck	40W
S30	66x26cm	PVC	-	Lumbar	110W
S40	57x46cm	PVC	-	Neck, shoulders and back	120W

HEATING UNDERBLANKETS

"DF" line underblankets are provided with detachable control unit and special fast-heating function.

Double underblankets are provided with two control units, one per side, permitting independent temperature control for each side.

MODEL	TYPE	SIZE (cm)	MATERIAL	HEATING ZONES	CONTROL UNITS	DETACHABLE	FAST HEAT	POWER (W)
U100DF	Single	130x70	Cotton	1	1	✓	✓	80
U110DF		150x80						150
UP110DF			Polyester					150
U200DF	Double	130x130	Cotton	2	2	✓	✓	2x80
U210DF		150x160						2x150
UP210DF			Polyester					2x150

INSTRUCTIONS

INSTALLATION INSTRUCTIONS FOR HEATING UNDERBLANKETS


Remove bedclothes and lay the underblanket directly on the mattress aligning it with the feet edge. The pillow side should remain uncovered.

The power cord will come out from the side, approximately at your shoulder level when lying down normally. Make sure the underblanket has no folds or creases. Check whether the power cord reaches the nearest mains socket. If necessary, you can turn the underblanket so that the power cord comes out at the feet side.

Do not fold the underblanket under the mattress. Lay a bottom bed sheet on the underblanket and proceed to make the bed as usual.

Since the underblanket lacks fixing elements, please make sure it does not get moved or creased while in use.

If possible, do not use your underblanket on adjustable beds or on twin mattresses; if this is the case, make sure the heating part and the power cord do not get nipped or damaged.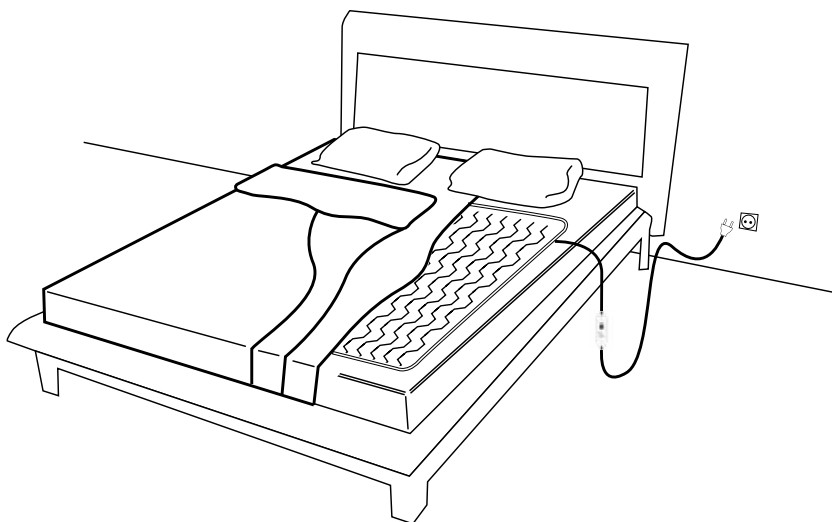

OPERATING INSTRUCTIONS

- 1) Plug the product into the mains socket, making sure your mains' specifications correspond to those of the product stated on the label. For products with detachable control unit, first connect the power cord to the product and then to the mains socket. For detachable products, use only the control unit originally provided with the product.
- 2) If everything is correct the control unit is then automatically set at the level 0 which indicates that the product is switched off. All LED indicators will go off.
- 3) Proceed to adjust the temperature among the four available settings by pushing the button on the control unit. At this point the product is running normally. The LED indicator corresponding to the temperature setting you have selected will remain lit. After a few minutes you will be able to perceive warmth. When using an "S" line heating pad, make sure to have adjusted and fastened the pad correctly onto a specific part of the body. Check that there are no creases or folds.
- 4) To switch off the product proceed to push the button until all the LED indicators are off (level 0). The heating element remains switched off in this position. To turn the product on push the button. Remember to unplug the product from the mains if you do not use it for a considerable period of time.
- 5) Unplug the product and let it cool down before folding it for storage.

ADDITIONAL INSTRUCTIONS FOR "S" LINE HEATING PADS

Thanks to its special shape and unique fasteners the "S" line heating pads conform perfectly to specific parts of your body for effective, targeted treatment.

The S20 heating pad (neck) offers an especially contoured flexible wrap, ideal for relief of muscle groups in the neck and cervical areas.

The S30 heating pad (lumbar) has been designed to fit the lumbar region.

The S40 heating pad (neck and back) is a new, unique pad design sized and shaped to conform to your neck, shoulders and upper back.


The S20 Model

Put the S20 pad round your neck and fasten it with the help of the fastener found on the cotton cover. If placed correctly, the pad will set just under your chin.

The S30 Model

Put the S30 pad around your waist and fasten it with the help of the elastic straps integrated into the pad's body. Elastic straps allow for easy fit to any waist size.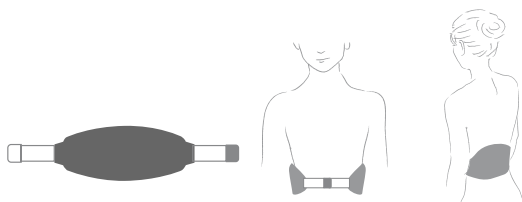

The S40 Model

Put the S40 pad on the upper part of your back and fasten the clasp you will find in the neck area. Then pass each of the two ribbons round the waist to fix the pad to your back.

SAFETY ELEMENTS

All our products are provided with a multiple safety system:

1) Digitally controlled electronic circuit.

The control unit was specifically designed and exhaustively tested to guarantee the highest safety standards.

Thanks to its exclusive high-precision electronic circuit, the control unit maintains the temperature of the heating element constant, as well as ensures the correct operation of the product. If a flaw such as excessive temperature is detected, the control unit will immediately switch off the heating element. If this happens, the digital temperature level indicator of the control unit will start flashing.

2) Safety AUTO-OFF.

To maximize safety when using the product, an additional safety switch off function has been incorporated into every control unit. This function sets a maximum period of time for which the product can operate continuously. The period is set for 2 hours and 12 hours for the heating pads and heating underblankets, respectively. When these periods have elapsed, and if the control button has not been pushed in the meantime, the product will switch off automatically. This reduces the risk to the user of spending too much time in contact with the heating element, as well as the risk of leaving the product in operation unnecessarily. When the safety AUTO-OFF function is activated, the heating element is switched off and the LED indicators of the controller go off. To restart the product push the control unit button.

3) Safety heating elements.

The heating elements of all our products use double-wound wiring with an integrated internal sensor element. They boast an especial technology that prevents hot spots and makes impossible the overheating of the element. If, through a product failure or improper use, the product reaches or exceeds a predetermined threshold value, the heating element will be automatically switched off, thus avoiding the risk of skin burns or fire.

ADVICE ON TEMPERATURE SELECTION

Excessive heat continuously applied to the skin may produce skin burns. The maximum temperatures our products can ever reach are those permitted by the current regulations.

The heating pads reach temperatures between 65°C and 75°C, while the heating underblankets, considering their intended application, will run at a lower temperature range reaching temperatures between 45°C and 55°C.

(Note: all measures were collected in laboratory conditions with products totally covered, according to procedures described in the EN60335-2-17 Standard).

It usually takes 30 to 45 minutes for the product to reach the maximum temperature.

The products provided with fast heating option reduce this time to between 10 and 15 minutes. If you perceive the heat emitted by your pad is insufficient, try the following: remove the cotton cover from the pad and the clothes from the part of the body to which the pad is to be applied and then apply the pad directly on the skin covering its other side with a towel. This will increase the heat transmission, as well as avoid heat dissipation.

You can set the control unit at the maximum level to obtain a faster warm up. Remember, however, that you should not use this setting continuously for more than one hour. After that time, we recommend that you lower the temperature gradually. Thanks to the safety AUTO-OFF option, it will also turn the product off after 2h or 12h of continuous running. For overnight or continuous use we recommend setting the unit at its lowest level (level 1).

Therapeutic benefits provided by these products have little to do with high temperatures and are based on a consistent, controlled low-level heat. If, after a considerable period of time, the product stops warming and gets cool, it may be due to the activation of the safety AUTO-OFF, in order to protect you from having excess heat applied to your skin.

pekatherm TECHNOLOGY

This product family is specially designed to ensure a smooth low-level heat which, apart from comfort, provides recognised pain relieving effects, since it dilates the blood vessels and enables muscular distension.

The control unit was designed and developed together with the heating elements to assure the best performance.

The control unit is provided with high technology electronic circuitry, able to continuously control and regulate the temperature of the heating element. It is also able to detect increases and decreases in ambient temperature and compensate for them, thus ensuring that the temperature of the heating element is constant. It means that your pad or underblanket will always reach the same temperature, regardless of ambient and use conditions.

All our products have increased power to ensure a faster heating. During the first few minutes of operation full power is applied to reach a selected temperature and, once obtained, the power application drops drastically to ensure low consumption.

INSTRUCTIONS

TROUBLESHOOTING TABLE

PROBLEM	SOLUTION
1. When the product is plugged in, the LED indicators do not light.	Check whether your mains socket has power; check whether the product is plugged in correctly. Unplug the product and wait a few seconds, then plug it in again. Push the button on the control unit.
2. After plugging the product in, the control unit fails to set at the default level (level 0).	The control unit may have found an error at the start and blocks the product operation for safety. Check if any of the LED indicators has any special flashing. Unplug the product from the mains, wait a few seconds and plug it in again. If the problem persists, take the product to an authorized service centre.
3. The product does not warm up.	Make sure the product is operating by checking whether a LED indicator of the control unit is lit. Remember that the safety AUTO-OFF function may have switched the product off if the safety time period is over. If all the LED indicators are off, push the button to switch the product on. In case of any doubt, reset the product by unplugging it from the mains and plugging it in after a few seconds. Remember that quick checks carried out with the product uncovered are not reliable. Try to cover the product with a blanket or a towel and wait at least 30 minutes. If you still do not perceive heat, take the product to an authorized service centre.
4. The product warms up insufficiently.	If, with the control unit set at the maximum level, you still perceive that the heat emitted by the product is insufficient, try the following: for heating pads, remove the cloth cover from the pad and the clothes from the part of the body to which the pad is intended to be applied and then apply the pad directly on the skin, covering its other side with a towel. This will increase heat transmission, as well as avoid the dissipation of some of the heat. In the case of heating underblankets, cover it with a blanket.
5. After a period of proper functioning, the product stops warming and all the LED indicators go off.	In order to ensure safety in use, all our products are provided with a safety AUTO-OFF option which switches the product off after a certain number of hours of continuous operation if the temperature button has not been pushed during this time. The periods are 2 hours for heating pads and 12 hours for heating underblankets. See SAFETY ELEMENTS paragraph for details. If you want to turn on the product, just push the button on the controller.
6. The control unit appears to malfunction; the LED indicators flash continuously.	The control unit may have found a problem and blocks the product operation for safety. Unplug the product from the mains, make sure there are no creases or folds, wait a few seconds and plug it in again. If the problem persists, take the product to an authorized service centre.

CLEANING

General precautions

Before cleaning the product remember to unplug it from the mains.

Heating pads and heating underblankets are washable.

Those models with detachable control unit can be hand or machine washed, after having detached the control unit.

Those with non-detachable control units can only be hand washed, always taking great care not to wet - or splash water on - the control unit.

The control unit is not washable. Prevent the control unit from getting wet or moist.

Do not dry clean the product. Do not use strong detergents or cleaning agents containing solvents.

Machine washing

For machine washing, select a programme for delicate garments, at 30° C.

Hand washing

If hand washed, use a soft detergent and rub or brush softly and always in the direction of the internal wiring. Never wring the product.

To get smaller stains out you may prefer using a cleaning foam instead of washing. Apply the foam regularly on the product's surface using a moist cloth or sponge, and then brush it in the direction of the internal wiring.


PVC pads can be easily washed using a moist cloth or sponge and then should be dried with a dry cloth.

Heating pads are provided with a removable cover which may be hand or machine washed. Remove the cover and follow the instructions found on the label inside the cover. When machine washed, it is recommendable to put the cover in a special bag for delicate garments.

Drying

After washing, the product should not be used in case of wet or damp. If the product gets wet, do not wring it out to dry it, do not use tumble dryers and never iron or switch the product on to dry it. To dry the product extend it flat or hang it on a line avoid folding it.

For products fitted with a non-detachable control unit, the control unit should be positioned on the clothesline to ensure that water could not enter inside it.

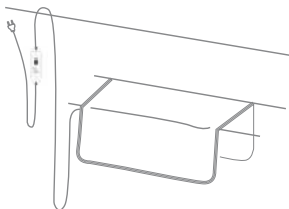INSTRUCTIONS

STORAGE

Let the product cool down before folding it for storage. Avoid excessive folds or creases. Store it in a cool dry place. Do not place any heavy objects on the product.

Inspect the product (the control unit, the heating elements and the power cord) frequently for signs of wear, damage or misuse. If these signs appear, do not use the product. Remember this product has no parts intended to be repaired or replaced by the user. Do not try to carry out repairs and, instead, take the product to an authorized service centre.

Repairs may only be carried out by our after sales service or at an authorized service centre. Improper repair may cause considerable danger to the user. In the event of any doubt, please contact our customer service.

ADVICE ON DISPOSAL

If you decide to dispose of this product, first make it inoperative. After unplugging it, cut the power cord off. It is also recommended to render other parts of the product inoperative in order to prevent children from playing with them.

According to EU Directive 2002/96/EC WEEE (Waste of Electrical and Electronic Equipment), electrical and electronic devices must be disposed of separately from household refuse, delivering them to a special refuse collection point, in order to assure the subsequent dispatch of the discarded equipment for recycling, treatment and environmentally compatible disposal, thus contributing to the prevention of possible negative effects on the environment and on health.

Your local authorities will provide you further information on how to dispose of your inoperative appliances.

WARRANTY

pekatherm S.L. guarantees this product against manufacturing and material defects for a period of three years from the date of purchase.

pekatherm S.L. will replace or repair the defective item free of charge, including labour and spare parts, if the defect derives from manufacturing and/or material reasons, and if the product is handed by the customer to an authorized service centre.

If replacement or repair is not possible, the customer may claim for refund or buy a superior model paying the difference.

This warranty will not be valid if not accompanied by the original purchase bill or ticket.

Application of the warranty will not imply any extension of its period.

This warranty will not cover any defect due to reasons beyond manufacturing, such as transportation damages, non observance of the operating instructions, careless or improper use, wear and tear, damages caused by a blow or fall, water, chemical agents, etc.

This warranty will not cover any cases of improper use of the product, such as professional or medical use.

This warranty will not cover any damage if improper repairs were attempted by unauthorized personnel.

pekatherm S.L. declines all responsibility for any damage caused directly or indirectly to persons, pets or properties deriving from non observance of these operating instructions.

This warranty is only valid in the country where the product was purchased.

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